

# Sterile Drill Cover: Surgery Preparation Instructions

## 1 Obtaining Sterile Equipment



Collect sterile parts:  
1. Drill cover  
2. Drill shaft  
3. Chuck key



## 2 Preparing Sterile Drill Cover



2.1. Roll drill cover edge back 2 times. Expose only inside lining of drill cover to non-sterile team member



2.2. Make sure sterile team member's gloves are tucked under fabric (→) and away from opening of the drill cover




## 3 Preparing The Drill



Install charged battery



Centre safety toggle

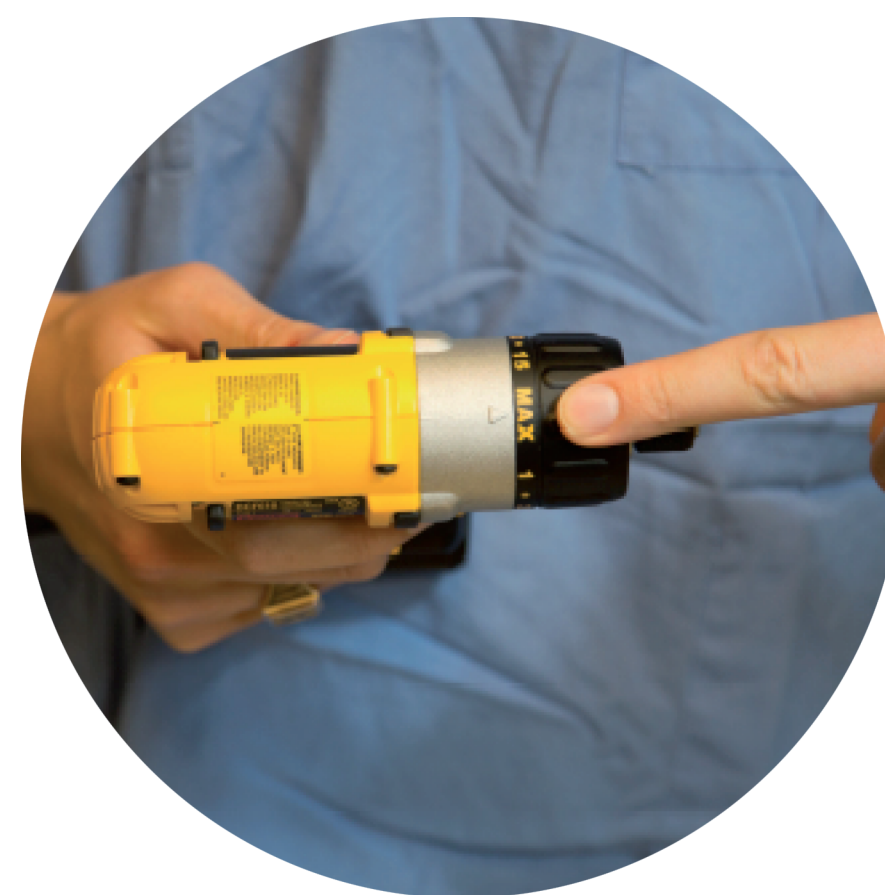

Set torque to MAX

## 4 Transferring The Drill



4.1. Sterile nurse presents the rolled up cover



4.2. Non-sterile nurse inserts drill into cover. Drill and non-sterile hand can only touch the inside fabric of the cover



## 5 Sealing The Drill Cover



5.1. Unfold drill cover and tightly roll drill cover opening 3 times



5.2. Buckle clip



## 6 Attaching The Drill Shaft



6.1. Pull back loose fabric, expose receiving end drill chuck. DO NOT push unsterile drill through opening (→)



6.2. Insert shaft into drill. You should feel a click. DO NOT remove shaft once inserted



6.3. Screw together chuck and drill cover until tight and red O-ring is no longer visible



## 7 Insert drill bit and tighten with chuck key

